


MQT Testing Guide

MQT Guide

1. Apply allergens and control solutions using a ComforTen® or QuinTip® device
2. Measure controls and allergens at 15-20 minutes
3. Use the flow chart to determine allergen dilution to apply by intradermal injection
4. Perform intradermal tests (IDT)
5. Mark and measure wheals at 10-15 minutes
6. Interpret the results and select endpoint

Note:

- If applying a #2 dilution, a similar dilution of glycerin should be applied as a negative control
- A vial safety test should be performed prior to initiating immunotherapy

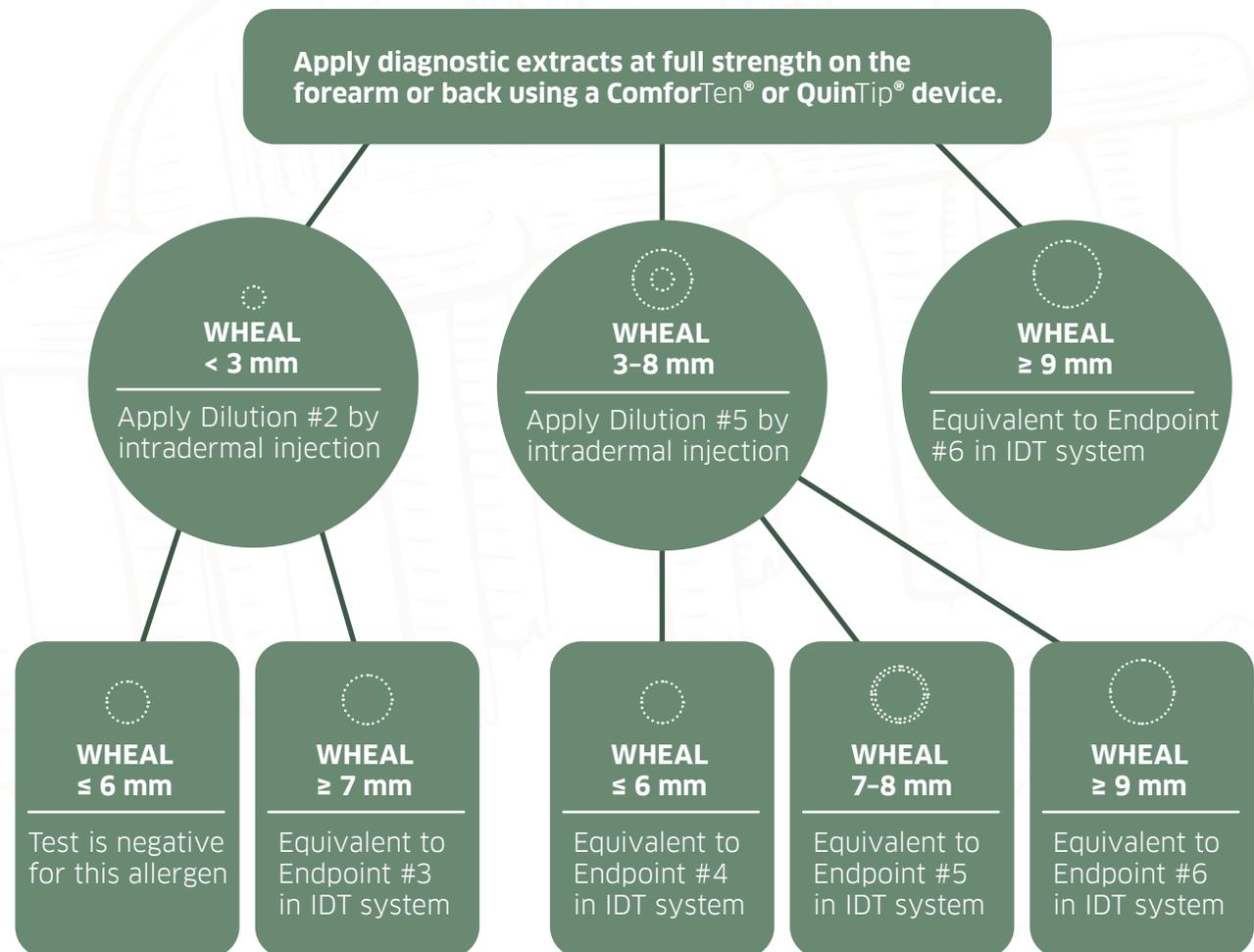

Adapted from: Quantitative Skin Testing for Allergy: IDT and MQT, 2006
 American Academy of Otolaryngic Allergy Clinical Care Statements, 2015
 Refer to the ComforTen® and QuinTip® package inserts for more information

Elevating Allergy Testing

Ensure reproducible, accurate testing results during diagnostic testing using our ComforTen® Skin Test System.¹


Both the ComforTen® Multiple Skin Test and QuinTip® Individual Skin Test devices integrate seamlessly within our testing system, offering precise results alongside our expert-recommended Histamine Positive Control.^{2,3}

References:

1. Nelson, H.S., et al. "Evaluation of Devices for Skin Prick Testing" *J. Allergy Clin. Immunol.*; 101: 153-156. February 1998
2. N.F. Adkinson, B. B. (2014). *Middleton's Allergy: Principles and Practice. 8th ed.*, 1123
3. R.B. Berkowitz, D.G. Tinkleman, D. Lurz, et al. Evaluation of the Multi-Test Device for Immediate Hypersensitivity Skin Testing. *J. Allergy Clin. Immunol.*; 92: 750-756, November 1993.



Request a quote and see how we make the switch to our ComforTen® Skin Test System easy.

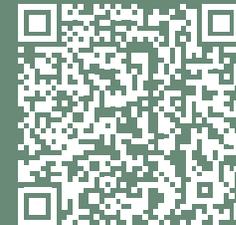

HollisterStier
Allergy