




MQT Guide

1. Apply allergens using a ComforTen® or Quintip® device.
2. Measure controls at 10 minutes.
3. Mark and measure wheals at 15 and 30 minutes.
4. Use the flow chart to determine intradermal dilutions.
5. Perform intradermal tests (IDT).
6. Mark and measure wheals at 10 minutes.
7. Interpret the results and select endpoint.

A vial safety test should be performed prior to initiating immunotherapy.
(Follow the vial safety procedure)

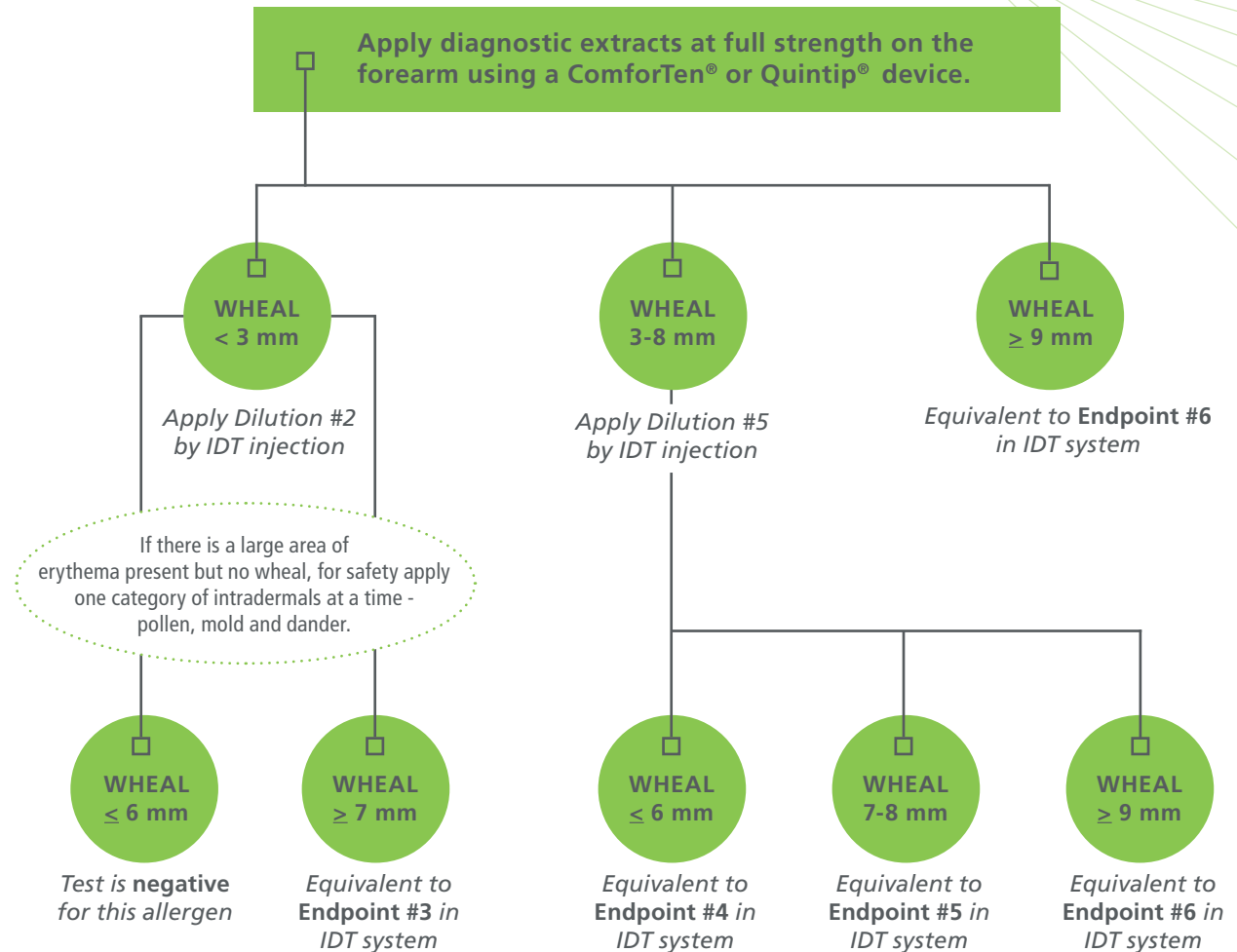

Adapted from:
Quantitative Skin Testing for Allergy: IDT and MQT, 2006
American Academy of Otolaryngic Allergy Clinical Care Statements, 2015
Refer to the ComforTen® and Quintip® package inserts for more information.



QUALITY. PURITY. RELIABILITY.



1.800.992.1120 | hsallergy.com
3525 North Regal Street | Spokane, WA 99207-5788